

Technical
Datasheet
08.08.2023
REV. 2.0

Bensan Instrulube by Safecare Instrument Lubricant

*Instrument
Lubricant
Concentrate and
Spray*



Application Areas

Safecare's Bensan Instrulube instrument lubricant is a non-silicone based lubricant rust inhibitor that extends the instrument life, protecting it from scale and corrosion while facilitating the cleaning process both manually using the ready-to-use product or in a washer disinfectant using the concentrate. The product is phosphate free.

Features:

- Ensures maximum protection for the instruments
- Prevents corrosion, discoloration and scale build-up
- Facilitates the cleaning process
- Maintains maximum tactile function
- Ensures proper operational condition of moving parts such as hinges
- Delivers up to 130 cycles per 4-liter canister of concentrate
- Fragrance Free Formulation

Composition

- White Mineral Oil <10%
- Surfactants & corrosion inhibitors <5%
- Other

Shelf Life

3 Years from the date of manufacture

Efficacy Claims

Form	Concentration
Concentrate	Dilution from 0.1% to 0.8%
Ready-To Use	Use Undiluted

Chemical-physical data

Color	Opaque White
Odour	Odourless
Storage	Store at 0°C to 30°C
Form	Liquid
pH	8-9

Toxicological Information

Carcinogenicity	None of the components of this product are listed as a carcinogen or suspected carcinogen by OSHA, IARC, and NTP
Chronic Effects	None known
Reproductive Effects	None known
Mutagenic effects	Not expected to cause mutagenic activity
Eye	Rinse with plenty of water
Skin	None known

Inhalation	None known (get medical attention in case of symptoms)
Ingestion	Seek medical attention if irritation persists

Best Practices

Always read the label and product information before use. Wearing protective personal equipment (e.g. made of nitrile rubber) is required. Do not mix with other cleaning agents.

Instructions for Use

Adjust the concentrate, contact time and water temperature according to the water quality and the cleaning methods following the instructions of the hardware manufacturer. For automatic use in a washer disinfector or manual application using the RTU product. European classification according to Medical Device Directive 93/42/CEE: Class I. Corresponds to EN norm 15883.

Order Information

Name	Unit of Measurement (UOM)	Reorder Number
Bensan Instrulube	2 X 4 Liters /case Concentrate	IRL1001

Related Products

- Bensan Quinzyme
- Bensan Pera
- Bensan OPA
- Bensan HyPo

Safecare Medical Industries manufactures economically attractive high-quality disinfectants and instrument reprocessing products following environmentally friendly manufacturing processes in our state-of-the-art facility.

Safecare Medical Industries

KIZAD KHIA 8-18
PO. Box 133685, Abu Dhabi
United Arab Emirates
Telephone: +971 2 506 7333

