Technical Datasheet

Bensan Omni Econo Wipes

Non-Alcoholic Disinfection Wipes

Alcohol-free
Surface
Disinfectant
Wipes



Bensan Omni Econo Wipes presoaked wipes are ready-to-use for hard, non-porous environmental surfaces as well as surfaces of non-critical medical devices that get in touch with the intact skin. The formulation can be applied on stainless steel, plastic, Formica® and glass among others. The formulation consists of a combination latest generation quaternary ammonium compounds and surfactants and offers excellent cleaning and disinfection efficacy against a wide range of pathogenic microorganisms within a 2 minute exposure time. The non-toxic formulation is compatible with a wide variety of medical devices and can be applied broadly including on alcohol-sensitive surfaces. Examples include general patient areas, endoscopy, Intensive Care Units, oncology facilities, emergency rooms, dialysis centers, The wipes can be used on any equipment that allows for a spray-and-wipe disinfection on high-touch surfaces which are compatible with alcohol. Example include blood pressure cuffs, IV polls, incubators, ultrasonic probes, dialysis units, patient beds, mattreces, patient monitors, neck collars, bed railings, bassinets, computer monitors and keyboards among others.





- Bactericidal, Virucidal, Fungicidal and Tuberculocidal in 2 minutes or less
- Excellent Material compatibility
- Alcohol-free, Phenol-free, aldehyde-free formulation
- Completely non-toxic
- Large coverage area / wipe highly cost-effective





Composition

- 0.33% Didecyl Dimethyl Ammonium Chlorides
- 0.12 Benzethonium Chloride
- Surfactants
- Excipients

Packaging Sizes

• 12 canisters per case

200 Economy Wipes/ Flowpack

Shelf Life

3 Years from the date of manufacture

Efficacy Claims

Efficacy	Concentration	Contact Time	EN Norm
Virucidal (enveloped)	Ready-to-use	1 Min.	EN 14476 Dirty Conditions
Virucidal (non- enveloped - Noro)	Ready-to-use	2 Min.	EN 14476 Dirty Conditions
Bactericidal	Ready-to-use	2 Min.	EN 13727, EN16615 Dirty Conditions
Fungicidal	Ready-to-use	1 Min.	EN 13624, EN16615 Dirty Conditions
Tuberculocidal	Ready-to-use	2 Min.	EN14348

Chemical-physical data			
Color	Colorless		
Odor	Solvent odor		
Flammability	Non-Flammable		
Form	Liquid		
рН	8-10		

Toxicological Information

Carcinogenicity	None of the components of this product are listed as a	
و پ	carcinogen or suspected carcinogen by OSHA, IARC, and NTP	
	caremogen of suspected caremogen by OSTIA, IARC, and IVII	
Chronic Effects	None known	
Reproductive Effects	Not expected from this product	
Mutagenic effects	Not expected to cause mutagenic activity	
Eye	Can cause mild eye irritation	
Skin	Non-irritating	
Inhalation	High concentrations may cause upper respiratory tract irritation	
Ingestion	Ingestion is unlikely for solid products. No adverse effects are	
	expected.	
	expected.	

Best Practices

Use disinfectants safely. Always read the label and product information before use. Wearing protective personal equipment (e.g. made of nitrile rubber) is required. Do not mix with other cleaning agents. Further information is available on request.

Order Information

Name	Unit of Measurement (UOM)	Reorder Number
Bensan Omni Econo Wipe	12 canisters / case (200 wipes each)	ICO2011

Related Products

- Omni Spray
- Premium Spray and Wipes

Safecare Medical Industries manufactures economically attractive high-quality disinfectants and instrument reprocessing products following environmentally friendly manufacturing processes in our state-of-the art facility.

Safecare Medical Industries

KIZAD KHIA 8-18 PO. Box 133685, Abu Dhabi United Arab Emirates Telephone: +971 2 506 7333

www.bensano.com





