

Technical  
Datasheet  
27.07.2023  
Rev. 3

# Bensan Hypo Spray

## Hydrogen Peroxide based Surface Disinfectant

*Intermediate Level  
Disinfectant based  
on Hydrogen  
Peroxide*



### Application Areas

Safecare's Bensan Hypo Spray is a ready-to-use intermediate level, sporicidal surface-disinfection solution for use on hard, non-porous environmental surfaces as well as surfaces of non-critical medical devices that get in touch with the intact skin. The formulation can be applied on stainless steel, plastic, Formica® and glass among others. The formulation consists of a combination of Hydrogen Peroxide, colloidal silver and surfactants and offers excellent residue-free cleaning and disinfection efficacy against a wide range of pathogenic microorganisms within a 2 minute exposure time. The non-toxic formulation which is compatible with a wide variety of medical devices can be applied broadly. Examples include general patient areas, endoscopy, dialysis centers, operating rooms or CSSD.

Use a lint free-cloth for an effective spray-and-wipe disinfection.

### Features

- Sporicidal, Virucidal, Bactericidal, Fungicidal/Yeastocidal
- Sporicidal in 5 Minutes
- Completely residue-free
- Excellent material compatibility
- Completely non-toxic, biodegradable, environmentally friendly
- Odorless
- Effectively removes mold and bad odors

### Composition

- 5% Hydrogen Peroxide
- <2% Peracetic Acid
- <1% Phosphoric Acid

### Shelf Life

18 months from date of manufacture

### Efficacy Claims

Efficacy	Concentration	Contact Time
Virucidal	Ready-to-Use	1 min. EN 14476
Bactericidal	Ready-to-Use	2 min. EN 13727,
Fungicidal	Ready-to-Use	2 min. EN 13624,
Tuberculocidal	Ready-to-Use	3 min. EN 14348
Sporicidal	Ready-to-Use	5 min. EN 17126

## Chemical-physical data

Color	Clear yellow
Density	>1
Odour	Slight Acetic Acid odor
Flash Point	Not flammable
Form	Liquid
pH	1.5-2.5

## Toxicological Information

Carcinogenicity	No known significant effects or critical hazards.
Chronic Effects	None known
Reproductive	No known significant effects or critical hazards.
Mutagenic effects	No known significant effects or critical hazards.
Eye	Can cause eye irritation category 2 (US)
Skin	Skin sensitization category 1B (US)- causes severe burns
Inhalation	May cause respiratory irritation
Ingestion	May cause burns to mouth, throat and stomach.

## Best Practices

Use disinfectants safely. Always read the label and product information before use. Wearing protective personal equipment (e.g. made of nitrile rubber) is required. Do not mix with other cleaning agents. Further information is available on request.

## Order Information

Name	Unit of Measurement (UOM)	Reorder Number
Bensan Hypo Spray	10 X 1L	ICH1003
Bensan Hypo Spray	2 X 4L	HDH1008

## Related Products

- Bensan Hypo
- Bensan Hypo Floor
- Bensan Hypo Plus

**Safecare Medical Industries** manufactures economically attractive high-quality disinfectants and instrument reprocessing products following environmentally friendly manufacturing processes in our state-of-the-art facility.

## Safecare Medical Industries

KIZAD KHIA 8-18  
PO. Box 133685, Abu Dhabi  
United Arab Emirates  
Telephone: +971 2 506 7333

