

# Bensan Asperfect


Bensan Asperfect is a 4-in-1 alkaline cleaning, descaling, disinfection and deodorizing concentrate based on Quaternary Ammonium Compounds and tensioactive agents. Bensan Asperfect removes organic and inorganic compounds effectively from the inside of suction lines while providing disinfection action.

**COMPOSITION:**

1 Liter of Bensan Asperfect contains 45 g of Didecyl DimethylAmmonium Chlorides, 55 g of Potassium Hydroxide, surfactants, corrosion inhibitors, additives and water to 1L

**STEP-BY-STEP INSTRUCTIONS:**

- Rinse the aspiration system by drawing in 2 litres of cold water
- Prepare at least one litre of solution for each dental unit: fill a recipient with water and then add 10ml of Bensan Asperfect perlitre of water
- Stir
- Siphon the solution through the aspiration tubes
- Pour at least 250 – 300 ml of disinfectant solution into the spittoon
- Leave to work for 15 minutes for a rapid disinfection or for at least 4 hours for the full kill claim
- At the end of the exposure time, rinse the aspiration system by drawing in 2 litres of cold water
- Repeat the reprocessing procedure daily (after hours) and in case of heavy use twice daily during the down time (over lunch and after working hours)

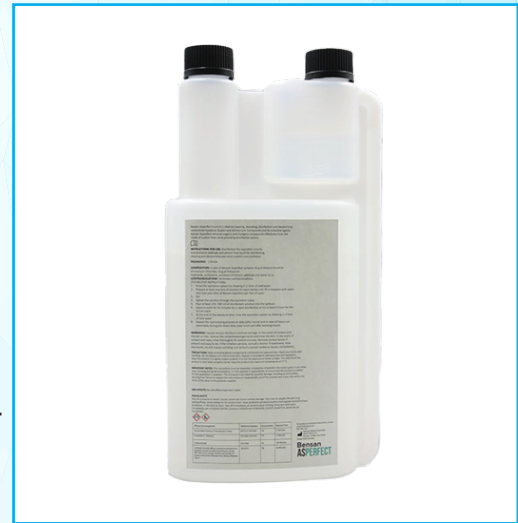

Efficacy (microorganisms)	Reference Standard	Concentration	Exposure Time
Bactericidal (S. Aureus, P. Aeruginosa, E. Hirae)	ENI3727, ENI4561	1%	15 Minutes
Fungicidal (C. Albicans)	ENI3624, ENI4562	1%	15 Minutes
Turberulocidal (M Terrae)	ENI4348	1%	240 Minutes
Limited virucidal efficacy towards enveloped and lipophilic viruses including blood borne viruses (HIV, HBV, HCV), Herpes simplex and families of virus such as the Influenza Virus, Ebola, Influenza Type A	ENI6777	1%	60 minutes

**Reorder Number: IRA1001**

For product or technical information, contact Safecare at +971-2-506733 or visit our website at [www.safecareind.com](http://www.safecareind.com).

KIZAD KHIA 8-18  
 PO. Box 133685, Abu Dhabi  
 United Arab Emirates



**Safecare**  
 MEDICAL