

BENSAN OPA



BENSAN OPA is a high-level disinfectant for reprocessing heat sensitive semi-critical medical devices for which sterilization is not suitable. When used according to the Directions for Use, Bensan OPA may be used or reused at or above its Minimum Recommended Concentration (MRC) of 0.3%, as determined by Bensan OPA test strips, in manual reprocessing with an immersion time of at least 12 minutes at a minimum temperature of 20°C, for a reuse period not to exceed 30 days. Bensan OPA solution may also be used or reused in a legally marketed Automatic Endoscope Reprocessor (AER) that can be set to a minimum temperature of 20°C, at or above its Minimum Recommended Concentration (MRC) as determined by Bensan OPA solution test strips, with an immersion time of at least 5 minutes at a minimum temperature of 25°C, for a reuse period not to exceed 28 days.

The formulation is highly compatible with all commonly used scopes, ultrasound units, and other semi-invasive medical devices.

Formulation Information:

0.6% Ortho-Phthalaldehyde, 2% Ethanol, sequestering agents, corrosion inhibitors

Efficacy Claims for High-Level Disinfection, Manual Use at 20°C

| | |
|------------------------------------|------------|
| Fungi/Yeast incl. C. Auris | 12 Minutes |
| Bacteria | 12 Minutes |
| Enveloped Viruses | 12 Minutes |
| Non-Enveloped Viruses (incl. Noro) | 12 Minutes |
| Mycobacterium Tuberculosis | 12 Minutes |
| Bacterial and fungal Spores | 60 Minutes |

- Compatible with a wide range of surface materials
- Free of phenols, bleach or aldehydes
- Excellent high-level disinfection function thanks to a cocktail of OPA and other active ingredients
- Does not require activation
- Non-toxic formulation
- Bensan OPA Test strips measuring the Minimum Required Concentration of 0.3% of OPA (100 strips/tub)




- Available in 4-Liter canisters
- Economical
- Developed and manufactured in the United Arab Emirates with highest quality ingredients

List of Uses

- Hospitals
- Endoscopy
- OBGYNs
- Central Sterilization Services
- Dental offices
- Exterior surfaces of anaesthesia machines and respiratory therapy equipment
- Laboratories
- Nursing homes
- Operating rooms
- Ophthalmic and optometric facilities
- Outpatient surgical centers
- Surgical centers
- Whenever a steam sterilization if not possible or available

| | |
|--------------------------------|---------|
| Reorder Number Bensen OPA | HDO1001 |
| Reorder Number Bensen OPA Test | HDO1003 |


 Safecare Medical Industries
 KIZAD KHIA 8-18
 PO. Box 133685, Abu Dhabi
 United Arab Emirates

For product or technical information, contact Safecare Medical at +971 2 506 7333 or visit our website at www.bensano.com