

Technical
Datasheet
26.07.2023
Rev. 3

Bensan Pera Spray

Peracetic Acid based Surface Disinfectant

*Surface
Disinfectant Spray
based on
Peracetic Acid*



Application Areas

Bensan Pera Spray is a ready-to-use, sporicidal surface disinfectant spray for professional use in the healthcare industry. The foaming Peracetic Acid is a highly efficacious surface disinfectant that kills bacteria, viruses, fungi, TB and spores in 2 minutes.

Areas of Use:

- CSSD
- Surgery Centers
- ICU's
- ER
- General Patient & Public Access Areas

Features

- Sporicidal, Virucidal, Bactericidal, Fungicidal/Yeasticidal in 2 minutes
- Completely residue-free
- Non-toxic, biodegradable

Composition

- Peracetic Acid 6.0
- Hydrogen Peroxide 7%
- Acetic Acid <20%
- Surfactants
- Corrosion inhibitors

Shelf Life

18 months from date of manufacture

Efficacy Claims

Efficacy	Concentration	Contact Time
Virucidal	RTU	2 min. EN 14476
Bactericidal	RTU	2 min. EN 13727
Fungicidal	RTU	2 min. EN 13624
Tuberculocidal	RTU	2 min. EN 14348
Sporicidal	RTU	2 Min. EN 17126

Chemical-physical data

Color	Clear yellow
Density	>1
Odour	Acetic Acid
Flash Point	>100°C
Form	Liquid
pH	1-1.5

Toxicological Information

Carcinogenicity	No known significant effects or critical hazards.
Chronic Effects	None known
Reproductive	No known significant effects or critical hazards.

Mutagenic effects	No known significant effects or critical hazards.
Eye	Can cause eye irritation category 2 (US)
Skin	Skin sensitization category 1B (US)- causes severe burns
Inhalation	May cause respiratory irritation
Ingestion	May cause burns to mouth, throat and stomach.

Best Practices

Use disinfectants safely. Always read the label and product information before use. Wearing protective personal equipment (e.g. made of nitrile rubber) is required. Do not mix with other cleaning agents. Further information is available on request.

Order Information

Name	Unit of Measurement (UOM)	Reorder Number
Bensan Pera Spray	10 X 1L	HDP1003

Related Products

- Bensan Pera

Safecare Medical Industries manufactures economically attractive high-quality disinfectants and instrument reprocessing products following environmentally friendly manufacturing processes in our state-of-the art facility.



Safecare Medical Industries

KIZAD KHIA 8-18
 PO. Box 133685, Abu Dhabi
 United Arab Emirates
 Telephone: +971 2 506 7333

