

# Bensan Hypo Floor

**Bensan Hypo Floor** is a floor cleaning disinfection concentrate. The solution consists of a highly efficacious Hydrogen Peroxide-based formulation, leaving no residue, film or unpleasant odors during or after use. The solution is completely biodegradable.



## Formulation Information:

7.5% Hydrogen Peroxide, 0.23% Peracetic Acid, surfactants, sequestering agents

## Effective against:

- Enveloped and Non-Enveloped Viruses
- Gram-positive and Gram-negative Bacteria
- Fungi/Yeast
- Tuberculosis
- Bacterial Endospores

## Kill Claim:

Dilution	Exposure Time	Kill Claim
1:15 (7%) 1 part of Bensan Hypo Floor / 15 parts of Water (For use on tiled or hard surface floor only)	20 Minute exposure time	Virucidal (enveloped / Non-Enveloped) Bactericidal (Gram positive / Gram Negative) Fungicidal Tuberculocidal Sporicidal (C. Diff)
1:40 (2.5%) part of 1 Bensan Hypo Floor / 40	30 Minute exposure Time	Virucidal (enveloped / Non-Enveloped) Bactericidal (Gram positive / Gram Negative) Fungicidal Tuberculocidal Sporicidal (C. Diff)

## Features:

- Strong concentrate
- Economical in use
- Free of phenols, bleach or aldehydes
- Excellent cleaning function
- Leaves no residue
- Non-toxic formulation
- Completely biodegradable
- Available in 4-Liter canisters
- Developed and manufactured in the United Arab Emirates with highest quality ingredients

## Directions for Use:

- Dilute 1 part of Bensan Hypo Floor in 15 parts of water.
- Apply solution on floor and leave surface wet during required exposure time (see table).
- In case of heavy contamination with soil, remove material with a first cleaning pass with Bensan Hypo Floor following up with a second disinfectant pass.

When using Bensan Hypo Floor on waxed or coated flooring, dilute to 2.5% as per table above.

## List of Uses:

- Hospitals
- Ambulance equipment surfaces
- Animal care facilities
- Correctional facilities
- Dental offices
- Emergency medical settings
- Emergency vehicles
- Exterior surfaces of anaesthesia machines and respiratory therapy equipment
- Isolation areas
- Laboratories
- Nursing homes
- Operating rooms
- Ophthalmic and optometric facilities
- Outpatient surgical centers
- Surgical centers
- Washrooms in general patient areas and public access areas



Items	Reorder Number	Content
Bensan Hypo Floor	HDH1005	4 Liters



Safecare Medical Industries  
KIZAD KHIA 8-18  
PO. Box 133685, Abu Dhabi  
United Arab Emirates

