

# Bensan IPA Spray by Safecare High-Alcohol Disinfection Spray

## Surface Disinfectant Spray

### Application Areas

Safecare's Bensan IPA high-level alcohol solution is a ready-to-use spray for use on hard, non-porous environmental surfaces as well as surfaces of non-critical medical devices that get in touch with the intact skin. The formulation can be applied on stainless steel, plastic, Formica® and glass among others. The formulation consists of a combination of Isopropanol and surfactants and offers excellent residue-free cleaning and disinfection efficacy against a wide range of pathogenic microorganisms within a 1 minute exposure time. The non-toxic formulation which is compatible with a wide variety of medical devices can be applied broadly. Examples include general patient areas, endoscopy, dialysis centers, operating rooms or CSSD.

The solution can be used on any equipment that allows for a spray-and-wipe disinfection and is compatible with high levels of alcohol.

### Composition

- 70% Isopropanol
- Surfactants

### Packaging Sizes

Metric	1-Liter bottle
--------	----------------

### Shelf Life

3 Years from the date of manufacture

### Efficacy Claims

Efficacy	Concentration	Contact Time	EN Norm
Virucidal	Ready-to-use	1 Min.	EN 14476 Dirty Conditions
Bactericidal	Ready-to-use	1 Min.	EN 13727, EN 16615 Dirty Conditions
Fungicidal	Ready-to-use	1 Min.	EN 13624, EN 16615 Dirty Conditions
Yeasticidal	Ready-to-use	1 Min.	EN 13624, EN 16615 Dirty Conditions
Tuberculocidal	Ready-to-use	1 Min.	EN 17126 Clean Conditions

### Chemical-physical data

Color	Colorless
Density	Approx. 0.97g / cm <sup>3</sup> at 20°C
Odour	Mildly alcoholic door (isopropanol)
Flash Point	Not applicable
Form	Liquid
pH	10.5-12.5

### Toxicological Information

Carcinogenicity	None of the components of this product are listed as a carcinogen or suspected carcinogen by OSHA, IARC, and NTP
Chronic Effects	None known
Reproductive Effects	Not expected from this product
Mutagenic effects	Not expected to cause mutagenic activity
Eye	Can cause eye irritation

Skin	No adverse effects are known
Inhalation	High concentrations may cause upper respiratory tract irritation
Ingestion	No adverse effects are expected

## Best Practices

Use disinfectants safely. Always read the label and product information before use. Wearing protective personal equipment (e.g. made of nitrile rubber) is required. Do not mix with other cleaning agents. Further information is available on request.


## Order Information

Name	Unit of Measurement (UOM)	Reorder Number
Bensan IPA Spray	10 X 1L / case	ICI1001

## Related Products

- Bensan Value Spray and Wipes
- Bensan Omni Spray and Wipes
- Bensan Premium Wipes
- Bensan Heavy Duty Spray and Wipes
- Bensan Quinzyme Enzymatic Wipes

**Safecare Medical Industries** manufactures economically attractive high-quality disinfectants and instrument reprocessing products following environmentally friendly manufacturing processes in our state-of-the-art facility.

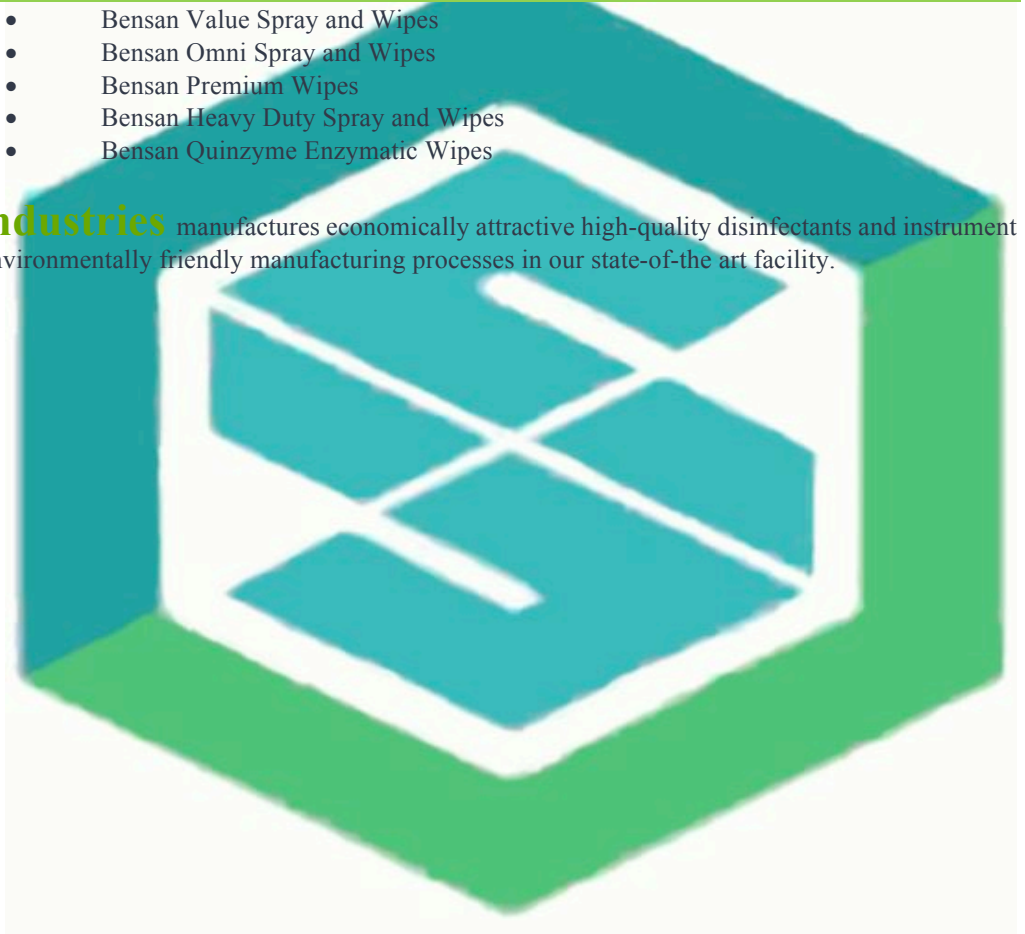

## Safecare Medical Industries

KIZAD KHIA 8-18  
 PO. Box 133685, Abu Dhabi  
 United Arab Emirates  
 Telephone: +971 55 433 4560

