

Bensan Instrulube

Safecare`s Instrulube Spray instrument lubricant milk is a non-silicone based ready-to-use lubricant rust inhibitor that extends the instrument life, protecting it from scale and corrosion while facilitating the manual cleaning process.

Formulation Information:

White Mineral Oil, Glycerine, Propylene Glycol, sequestering agents, excipients

FEATURES

- Ensures maximum protection for the instruments
- Does not interfere with the sterilization processes
- Prevents corrosion, discoloration and scale build-up
- Facilitates the cleaning process
- Maintains maximum tactile function
- Ensures proper operational condition of moving parts such as hinges
- For automatic or manual use
- Delivers up to 130 cycles per 4-liter canister of concentrate
- Fragrance Free Formulation


INSTRUCTIONS FOR USE:

- Shake well before using.
- Spray instruments after cleaning (box lock open), and before autoclaving
- Let instrument drain.
- Wipe off excess liquid.
- For best results, use Bensan Quinzyme or Bensan Sparkle instrument detergent.
- Note: Bensan Instrulube is not a detergent or sterilant.
- Instruments must be autoclaved after treatment.



LIST OF USES

- Hospitals including Central Sterile Supply Departments, Gastroenterology, Emergency Rooms
- Dental offices
- Outpatient surgical centers
- Surgical centers

☐  ☐ Safecare Medical Industries
KIZAD KHIA 8-18
PO. Box 133685, Abu Dhabi
☐ ☐ United Arab Emirates



For product or technical information, contact Safecare Medical at +971 2 506 7333 or visit our website at www.bensano.com