

Bensan Glow Spray

By Safecare Medical Industries

*Scale and rust
removing spray*



Application Areas

Bensan Glow Spray is a descaler and rust remover. This two-in-one product increases the life and durability of the instruments by removing rust, scale, spotting and inorganic deposits from the stainless steel instruments, in washer disinfectors or in autoclaves. The product is designed for a spray-and-wipe process, leaving no residue.

Features

- Ensures maximum protection for the instruments
- Does not interfere with the sterilization processes
- Removes rust, scale, spots, deposits from stainless steel instruments
- Facilitates the cleaning process
- Ensures proper operational condition of moving parts such as hinges
- Fragrance Free Formulation
- Developed and manufactured in the United Arab Emirates

Composition

- Phosphoric Acid <6%
- Glycolic Acid <2%
- Citric Acid <1%
- Excipients

Shelf Life

3 years from manufacturing date

Dilution

Ready-to-use solution

Chemical-physical data

Color	Clear
Odour	Acid smell
Storage	At room temperature in a dry, well ventilated area. Keep away from incompatible materials
Form	Liquid
pH	1.7-2.2



Best Practices

Use detergents safely. Always read the label and product information before use. Wearing protective personal equipment (e.g. made of nitrile rubber) is required. Do not mix with other cleaning agents.

Instructions for Use

Spray Bensan Glow Spray directly onto the surface and wait 30 seconds for the solution to act

Wipe surface with a lint-free non-woven substrate, removing loose scale, corrosion and debris from the surfaces

Let surfaces air-dry. For sensitive equipment, use plain water and a lint free cloth to wipe down surfaces after use of Bensen Glow Spray. Corresponds to EN norm 15883.

Order Information

Name	Unit of Measurement (UOM)	Reorder Number
Bensen Glow Spray	10 X 1L /Case	IRG2001

Related Products

- Bensen Glow Concentrate
- Bensen Lubricant Milk
- Bensen OPA
- Bensen HyPo
- Bensen Pera
- Bensen Sparkle

Safecare Medical Industries manufactures economically attractive high-quality disinfectants and instrument reprocessing products following environmentally friendly manufacturing processes in our state-of-the-art facility.



Safecare Medical Industries

KIZAD KHIA 8-18
 PO. Box 133685, Abu Dhabi
 United Arab Emirates
 Telephone: +971 2 506 7333

www.bensano.com

