

Technical  
Datasheet  
09.08.2023  
Rev. 1.0

# Bensan Omni Dry

## Concentrated Rinsing Solution and Drying Aid

*Instrument  
Rinsing and  
Drying Aid  
Concentrate*



### Application Areas

Bensan Omni Dry is a concentrated rinsing and drying aid for use in washer disinfectors. Bensan Omni Dry is a highly concentrated product, ensures the rapid drying of crockery and glassware to a streak-free, sparkling finish and prevents spotting. The product is added in the final stage of the washing cycle.

#### FEATURES:

For rapid, residue-free drying during automated reprocessing of surgical instruments, incl. ophthalmological instruments, anaesthesia utensils, sterilisation containers, implants as well as baby bottles.

Neutralises carried over alkali residues and reduces the formation of stains when using softened water in the final rinse cycle.

Excellent wetting properties on all surface materials

Reduced the formation of stains

Reduced the drying time

### Composition

- <10% Ethanol
- <10% Polyethylene Glycol
- <10% Sodium Cumene Sulphonate
- Excipients

### Shelf life

3 Years from date of manufacture

### Chemical-physical data

Color	Colorless
Odor	Solvent odor
Flammability	67°C
Form	Liquid
pH	9-11

### Dilution:

For surgical instruments, implants, anaesthetic equipment	0.2% - 0.4%
Bedframes	0.3% - 0.6%

### Toxicological Information

Carcinogenicity	None of the components of this product are listed as a carcinogen or suspected carcinogen by OSHA, IARC, and NTP
Chronic Effects	None known
Reproductive Effects	Not expected from this product

Mutagenic effects	Not expected to cause mutagenic activity
Eye	Can cause mild eye irritation
Skin	Non-irritating
Inhalation	High concentrations may cause upper respiratory tract irritation
Ingestion	No adverse effects are expected.

## Best Practices

Use disinfectants safely. Always read the label and product information before use. Wearing protective personal equipment (e.g. made of nitrile rubber) is required. Do not mix with other cleaning agents. Further information is available on request.

## Order Information

Name	Unit of Measurement (UOM)	Reorder Number
Bensan Omni Dry	2 X 4 L / case	IRO1001

## Related Products

- Bensan Quinzyme
- Bensan Sparkle
- Bensan Glow

**Safecare Medical Industries** manufactures economically attractive high-quality disinfectants and instrument reprocessing products following environmentally friendly manufacturing processes in our state-of-the-art facility.



## Safecare Medical Industries

KIZAD KHIA 8-18  
 PO. Box 133685, Abu Dhabi  
 United Arab Emirates  
 Telephone: +971 2 506 7333

