# Bensan Hypo Plus



Bensan Hypo Plus is a Ready-to-Use Environmental Surface Disinfectant for Use in a Fumigator and based on Hydrogen Peroxide.

### **Formulation Information:**

12% Hydrogen Peroxide, 0.23% Peracetic Acid, surfactants, sequestering agents

### **Effective against:**

- Enveloped and Non-Enveloped Viruses
- Gram-positive and Gram-negative Bacteria
- Fungi/Yeast
- Bacterial Endospores

## **Features:**

- 2-in-1 deodorizer disinfectant
- Non-toxic
- Alcohol-free, QAC-free, phenol-free, chlorine-free
- Completely Biodegradable
- No PPE required during use
- Safe for use during public access
- Formulated with <12% of boosted Hydrogen Peroxide

#### **INSTRUCTIONS FOR USE IN A FUMIGATOR**

- Ensure all surface areas including floors and walls are visibly clean
- Pour content of bottle into the holder of the fumigator
- Do not dilute
- Turn on fumigator and run vaporization until the surface areas are fully exposed in Bensan
- Hypo Plus according to size of room and surface area. Add more solution into the container as needed.
- Let surfaces air-dry. Do not wipe dry.
- Follow instructions for use of manufacturer to ensure sufficient fumigation of the area

#### **List of Uses:**

- Hospitals
- Ambulance equipment surfaces
- Animal care facilities
- Correctional facilities
- Dental offices
- Emergency medical settings
- Emergency vehicles
- Exterior surfaces of anaesthesia machines and respiratory therapy equipment
- Isolation areas
- Laboratories
- Nursing homes
- Operating rooms
- Ophthalmic and optometric facilities
- Outpatient surgical centers
- Surgical centers
- Washrooms in general patient areas and public access areas

ltems	Reorder Number	Content
Bensan Hypo Plus	HDH1004	4 Liters
Bensan Hypo Plus	HDH1006	1 Liter



Safecare Medical Industries KIZAD KHIA 8-18 PO. Box 133685, Abu Dhabi United Arab Emirates



For product or technical information, contact Safecare Medical at +971 2 506 7333 or visit our website at www.bensano.com