

Bensan IPA Wipes

Alcoholic Disinfection Wipes

*Alcohol-based
Surface
Disinfectant
Wipes*

Application Areas

Bensan IPA Wipes presoaked wipes are ready-to-use for hard, non-porous environmental surfaces as well as surfaces of non-critical medical devices that get in touch with the intact skin. The wipes can be applied on stainless steel, plastic, Formica® and glass among others. The formulation consists of a combination Isopropyl Alcohol and surfactants and offers excellent cleaning and disinfection efficacy against a wide range of pathogenic microorganisms within a 1 minute exposure time.

The non-toxic formulation is compatible with a wide variety of medical devices and can be applied broadly. Examples include general patient areas, endoscopy, Intensive Care Units, oncology facilities, emergency rooms, dialysis centers, touch surfaces which are compatible with alcohol. Example include:

- blood pressure and neck cuffs
- IV polls
- ultrasonic probes
- dialysis units
- patient beds, mattresses
- patient monitors, neck collars
- bed railings
- computer monitors
- keyboards among others

Features

| |
|--|
| • Bactericidal, Virucidal, Fungicidal and Tuberculocidal in 1 minute or less |
| • Excellent Material compatibility |
| • Phenol-free, aldehyde-free formulation, bleach-free |
| • Completely non-toxic |
| • Developed and Manufactured 100% in the UAE |
| • Large coverage area / wipe – highly cost-effective |

Composition

| |
|-------------------------|
| • 70% Isopropyl Alcohol |
| • Anionic surfactant |
| • Cleaning Agents |
| • Aqua |

Shelf Life

3 Years from the date of manufacture

Efficacy Claims

| Efficacy | Concentration | Contact Time | EN Norm |
|---------------------------|---------------|--------------|-------------------|
| Virucidal (enveloped) | Ready-to-use | 1 Min. | EN 14476 |
| Virucidal (non-enveloped) | Ready-to-use | 1 Min. | EN 14476 |
| Bactericidal | Ready-to-use | 1 Min. | EN 13727, EN16615 |
| Fungicidal | Ready-to-use | 1 Min. | EN 13624, EN16615 |
| Tuberculocidal | Ready-to-use | 1 Min. | EN14348 |

Chemical-physical data

| | |
|--------------|-----------------|
| Color | Colorless |
| Odor | Alcoholic odor |
| Flammability | Flammable |
| Form | Saturated Wipes |
| pH | 8-10 |

Toxicological Information

| | |
|----------------------|--|
| Carcinogenicity | None of the components of this product are listed as a carcinogen or suspected carcinogen by OSHA, IARC, and NTP |
| Chronic Effects | None known |
| Reproductive Effects | Not expected from this product |
| Mutagenic effects | Not expected to cause mutagenic activity |
| Eye | Can cause eye irritation |
| Skin | Non-irritating |
| Inhalation | May cause upper respiratory tract irritation |
| Ingestion | Ingestion is unlikely for solid products. No adverse effects are expected. |

Best Practices

Use disinfectants safely. Always read the label and product information before use. Wearing protective personal equipment (e.g. made of nitrile rubber) is required. Do not mix with other cleaning agents. Further information is available on request.

Order Information

| Name | Unit of Measurement (UOM) | Reorder Number |
|------------------|--------------------------------------|----------------|
| Bensan IPA Wipes | 10 canisters / case (100 wipes each) | ICI2001 |

Related Products

- Bensan IPA Spray

Safecare Medical Industries manufactures economically attractive high-quality disinfectants and instrument reprocessing products following environmentally friendly manufacturing processes in our state-of-the art facility.

Safecare Medical Industries

KIZAD KHIA 8-18
 PO. Box 133685, Abu Dhabi
 United Arab Emirates
 Telephone: +971 2 506 7333
www.bensano.com

