

Technical
Datasheet
01.07.2023
Rev. 1.0

Bensan Bleach Econo Wipes

Bleach Based Disinfection Wipes

Surface Disinfectant
Wipes based on
Sodium
Hypochlorite



Application Areas

Bensan Bleach Econo Wipes are presoaked, ready-to-use wipes for use on hard, non-porous environmental surfaces as well as surfaces of non-critical medical devices that get in touch with the intact skin. The formulation can be applied on stainless steel, plastic, Formica® and glass among others. The bleach formulation offers excellent cleaning efficacy, allowing for a thorough disinfection in one pass within a 3 minute contact time (1 minute for viruses and bacteria). The wipes which is compatible with a wide variety of medical devices and can be applied broadly. Examples include general patient areas, endoscopy, dialysis centers, operating rooms, CSSD or neonatal wards.

compatible with bleach. Example include TEE probes, incubators, ultrasound probes, dialysis units, patient beds, mattresses, patient monitors among others.

The product is free of aldehydes and phenols.

Composition

- 7000PPM of Sodium Hypochlorite
- Solvents
- Surfactants
- Corrosion inhibitors

Packaging Sizes

10 Canisters/	80 Wipes/Canister
---------------	-------------------

Shelf Life

18 Months from the date of manufacture

Efficacy Claims

Efficacy	Contact Time
Virucidal (enveloped)	1 Min. (EN 14476)
Virucidal (non-enveloped)	3 Min. (EN 14476)
Bactericidal	1 Min. (EN13727)
Fungicidal	1 Min. (EN 13624)
Tuberculocidal	3 Min. (EN 14348)
Sporicidal	3 Min. (EN 17126)

Chemical-physical data

Color	Colorless
Odor	Bleach smell
Flash Point	Not determined
Form	Wipes (Solid)
pH	10.5-12.5

Toxicological Information

Carcinogenicity	None of the components of this product are listed as a carcinogen
-----------------	-------------------------------------------------------------------

	or suspected carcinogen by OSHA, IARC, and NTP
Chronic Effects	None known
Reproductive Effects	Not expected from this product
Mutagenic effects	Not expected to cause mutagenic activity
Eye	Can cause eye irritation
Skin	No adverse effects are known
Ingestion	No adverse effects are expected

Best Practices

Use disinfectants safely. Always read the label and product information before use. Wearing protective personal equipment (e.g. made of nitrile rubber) is required. Do not mix with other cleaning agents. Further information is available on request.

Order Information

Name	Unit of Measurement (UOM)	Reorder Number
Bensan Bleach Econo Wipes	10 canisters/case	BBW1080

Related Products

- Bensan Value Wipes
- Bensan Omni Spray and Wipes
- Bensan Premium Spray and Wipes

Safecare Medical Industries manufactures economically attractive high-quality disinfectants and instrument reprocessing products following environmentally friendly manufacturing processes in our state-of-the art facility.



Safecare Medical Industries

KIZAD KHIA 8-18
 PO. Box 133685, Abu Dhabi
 United Arab Emirates
 Telephone: +971 2 506 7333

