

BENSAN ANTIFOG

Bensan Antifog with Pad is a single use, non-sterile device that is designed to apply an antifogging solution to the scope when used prior to and during procedures that utilize an endoscope or laparoscope. Included in the Bensan Antifog kit is the Antifog bottle with solution and a foam pad with an adhesive bottom.

Therapeutic Indications:

The Antifog with Pad is a single use, non-sterile device to be used prior to and during endoscopic and laparoscopic procedures to prevent fogging of the scope lens.

Features and Benefits:

- Designed to help prevent the scope or camera lens from fogging during laparoscopic, gastroscopic and arthroscopic surgeries
- Can also be used on protective eyewear, goggles and glasses
- Nontoxic and laboratory tested to be safe for use within a body cavity
- Includes 10mL bottle with screw cap and adhesive-backed foam pad with x-ray detectable or radiopaque strip
- Not made with natural rubber latex
- Single use
- Developed and manufactured in the United Arab Emirates

Ingredients: Isopropyl Alcohol, Sodium Cumenesulfonate, Glycerin, Sodium Lauryl Ether Sulphate, Citric Acid, Water

Instructions for Use:

1. Dispense pouch contents onto sterile field.
2. Remove paper backing from foam pad and adhere foam pad onto sterile drape in a convenient location.
3. Remove cap and apply 5 or 6 drops of Anti-Fog solution to sponge.
4. Touch distal end of scope to sponge, blot dry (do not wipe dry) with sterile sponge.
5. Alternatively, apply solution directly to laparoscope lens.
6. Repeat as needed.


Shelf Life: 36 Months

Product Name	Reorder Number	UOM
Bensan Antifog	AVG1001	100 X 10mL Bottles



United Arab Emirates

Safecare Medical Industries
KIZAD KHIA 8-18
PO. Box 133685, Abu Dhabi

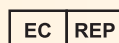

CMC Medical Devices
C/Horacio Lengo Nº 18
CP 29006, Málaga-Spain



Product Made in the United Arab Emirates

Reorder Number: **AVG1001**

Content: 50 bottles of 10mL