

Bensan Spiritus

Spirit-based Dual-Use Quick-Action Skin and Surface Disinfectant Solution

Bensan Spiritus is an Ethanol- / Methanol-based Quick-Action Skin and Surface Disinfectant solution.

Intended uses include:

- Disinfection of infusion or injection points
- Treatment of minor cuts and bruises
- Quick-action hand sanitizer
- Cleansing of instrumentation prior to surgical procedures
- General environmental surface disinfectant

Composition:

100g of solution contain:

Ethanol:..... 68.6g
Methanol:..... 37g
Inert Ingredients:... 27.7g



Instructions for Use as a Wound Treatment:

Spray Bensan Spiritus wound care solution directly onto the affected area. If needed, cover affected area with a plaster or cotton bandage held in place with bandage tape. Use Bensan Spiritus solution when replacing the bandage or as required.

Instructions for Use as a Surface Disinfectant:

Spray Bensan Spiritus directly onto pre-cleaned surface, thoroughly wetting area to be disinfected. Allow surface to remain visibly wet for 1 minute at room temperature. Let surface air dry.

Packaging Sizes

500mL bottle	5L Canister
--------------	-------------

Order Information

Name	Unit of Measurement (UOM)	Reorder Number
Bensan Spiritus	20 X 50mL / case	BSS1001
Bensan Spiritus	2 X 5L /case	BSS1002

Keep out of Reach of Children

Other information

- Keep Cool. Protect from Sunlight.
- store between 8°C and 15°C
- for additional information, see Safety Data Sheet (SDS)
- Discard packaging with care after use



Safecare Medical Industries
KIZAD KHIA 8-18
PO. Box 133685, Abu Dhabi
United Arab Emirates



CMC Medical Devices
C/Horacio Lengo N° 18
CP 29006, Málaga-Spain



Product Made in the United Arab Emirates

Reorder Number: BSS1001

Content: 500mL